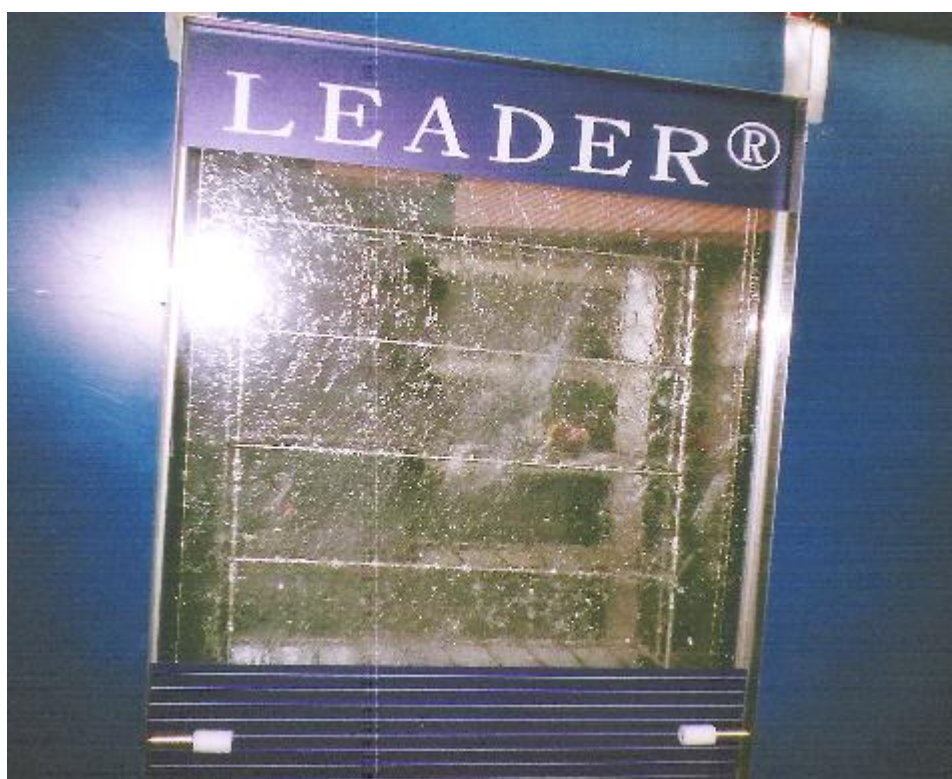
LEADER® MULTICHAMBER MEDICAL TUNNEL WASHER

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TEL : 02-2658-7810 FAX : 02-2658-7801

E-mail : leador.washer@msa.hinet.net

LEADER® MULTICHAMBER MEDICAL TUNNEL WASHER LEADER® LD-570T

The LEADER® LD-570T are especially designed for the CSR and offers due to the modular construction high efficient cleaning and best reprocessing results.

The machine can be made fit for purpose out of 1 – 4 independent elements:

Cleaning-, Disinfection-, Ultrasound- and Drying units.

LEADER® LD-570T

Temperature

Thermal disinfection

Good mechanical efficient Cleaning

Pump spray and ultrasonic cleaning

Time

Wash time and process °

Chemical Dosing system

Four liquid dosing pumps

LEADER® LD-570T :

• Key features

- > Up to 15 instrument trays [485x250x50mm]
- > RS232 interface for printer- or PC connection
- > Touch Monitor operator panel with 40 adjustable cleaning cycles
- > Automatic powered full glass door (s)
- > Washing and thermal disinfecting temperature adjustable up to [93°C]
- > Temperature monitoring by two independent PT-100 probes
- > Exhaust steam condenser optional °

Range of Use:

The machine is designed for cleaning the following parts:

1. glassware
2. drums and containers
3. surgical instruments.
4. MIS-Instruments
5. Rigid Endoscopes
6. Surgical Shoes
7. Ophthalmology
8. Anesthesia

LEADER ULTRASONIC CO., LTD.

LEADER Medical Washer Disinfector

LEADER® LD-570T

**BASIN BASKET AND
INSTRUMENT BASKET (Up
to 15 DIN trays)**

Extensive variants of equipment

- One-, Two-, Three-, Four-, module unit, vertical sliding doors
- Ultrasonic cleaning chamber
- Steam or electrical heating
- Monitoring of pump pressure, temperature, conductivity etc.
- Use of hollow jets with contact coupler to the basket (GMP rinse)
- High capacity HEPA filtered drying system
- Barcode reader for automatic basket identification
- Wide variety of baskets, customer specific manufacture
- Extensive automation, conveyors etc.
- Execution in ex-protection

**glass door allow visual inspection
of wash spray operation**

LEADER ULTRASONIC CO., LTD.

LEADER Medical Washer Disinfector

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LEADER ULTRASONIC CO., LTD.

LEADER Medical Washer Disinfector

LEADER® LD-570T

Cold Water (1)**

3/4" NPT, 2-5 bar. Coldwater must be supplied at 16°C minimum. Flow rate - 25 L/min

Warm Water (2)**

3/4" NPT, 2-5 bar. Hotwater must be supplied at (43°C) minimum. Flow rate - 25 L/min.

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3/4" NPT, (2-5 bar. Maximum flow rate - 25 L/min. Recommended minimum specific resistivity of 0.1 megohm per cm.

Drain (4)**

1-1/2" NPT. A 2" (51 mm) OD floor drain is recommended.

Electricity (5)**

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Steam (6)** †

1/2" NPT, 3-5 bar
Consumption - 45-60 lb/hr. Peak flow rate - 220 lb/hr.

Condensate Return (7)** †

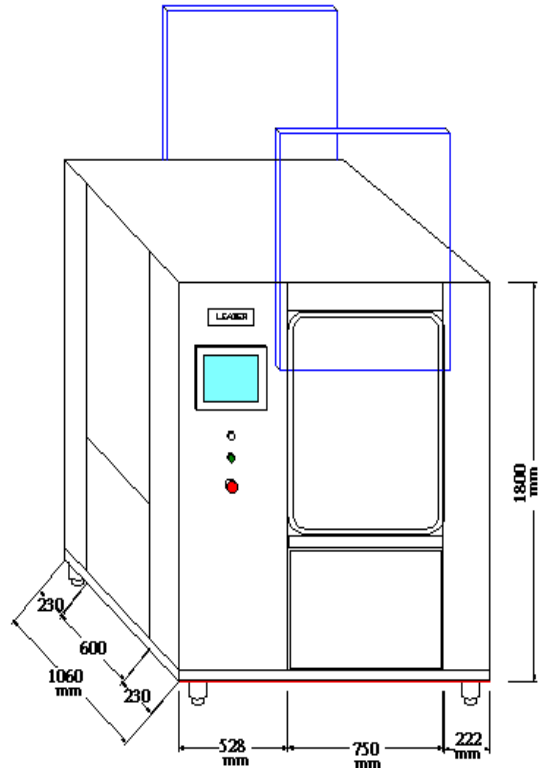
1/2" NPT. Peak - 0.4 gpm.

Air (9)†

1/4" NPT (8 mm), 44-87 psig (4-6 bar)
oil-free compressed air

Ventilation (8)**

3.5" (89 mm) OD vent connection.
Maximum flow rate - 206 scfm (350 m3/h).
Maximum Temperature 194°F (90°C).
6" (152 mm) if condenser backflow preventer is ordered.

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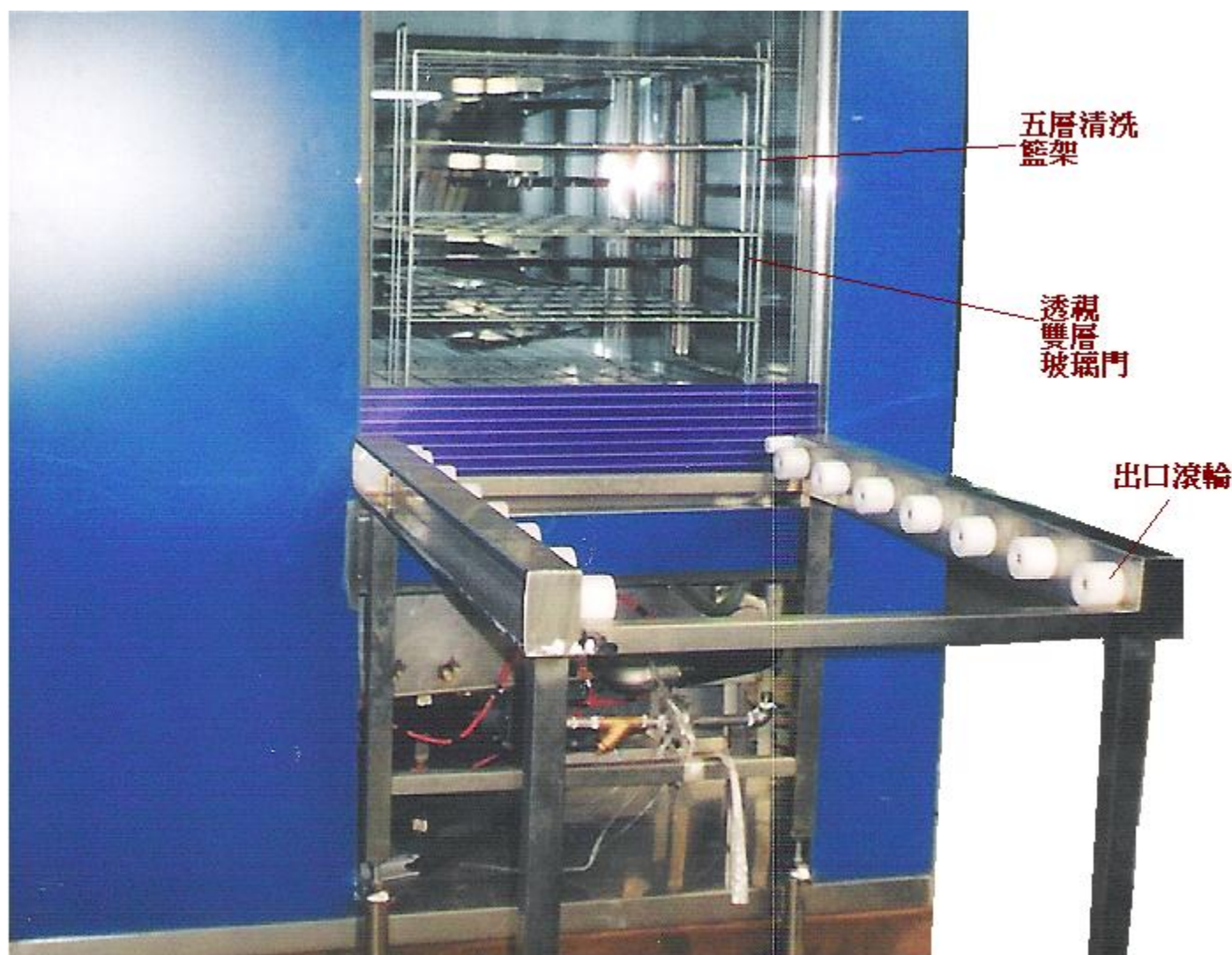
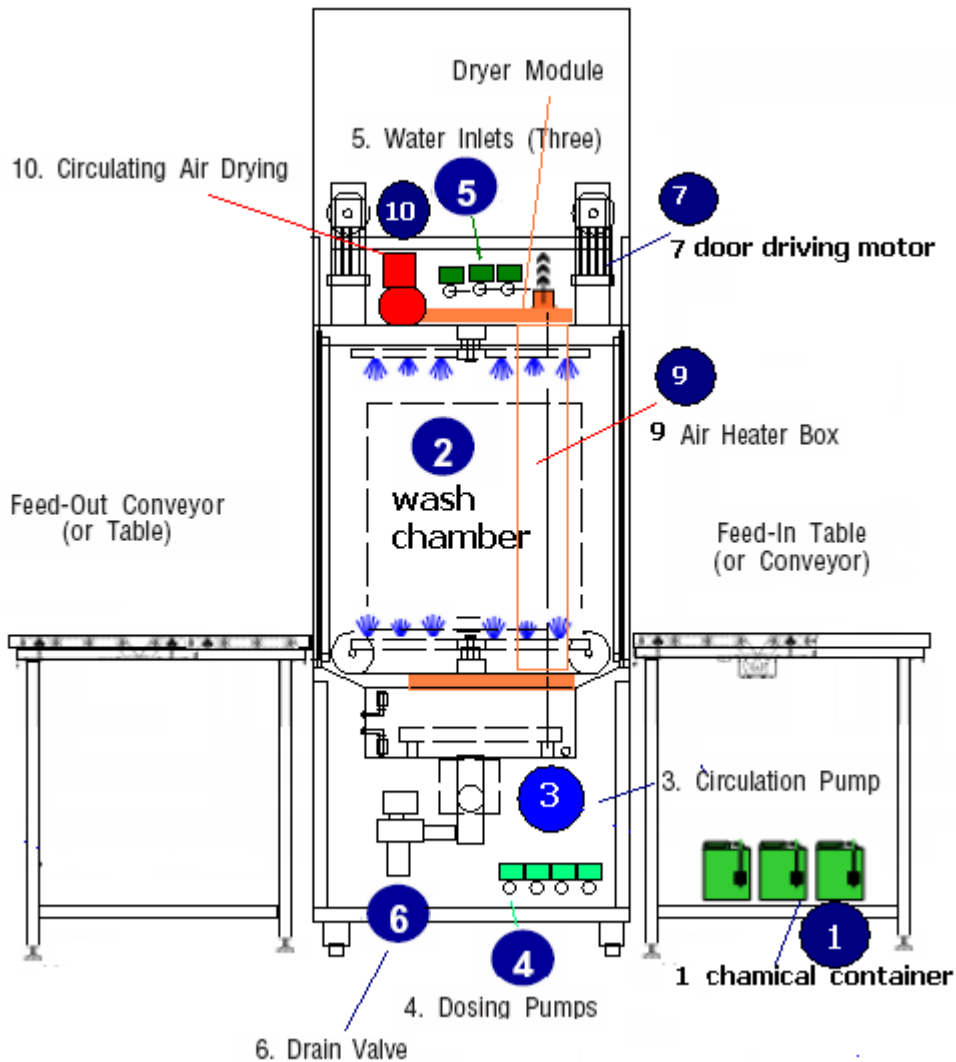
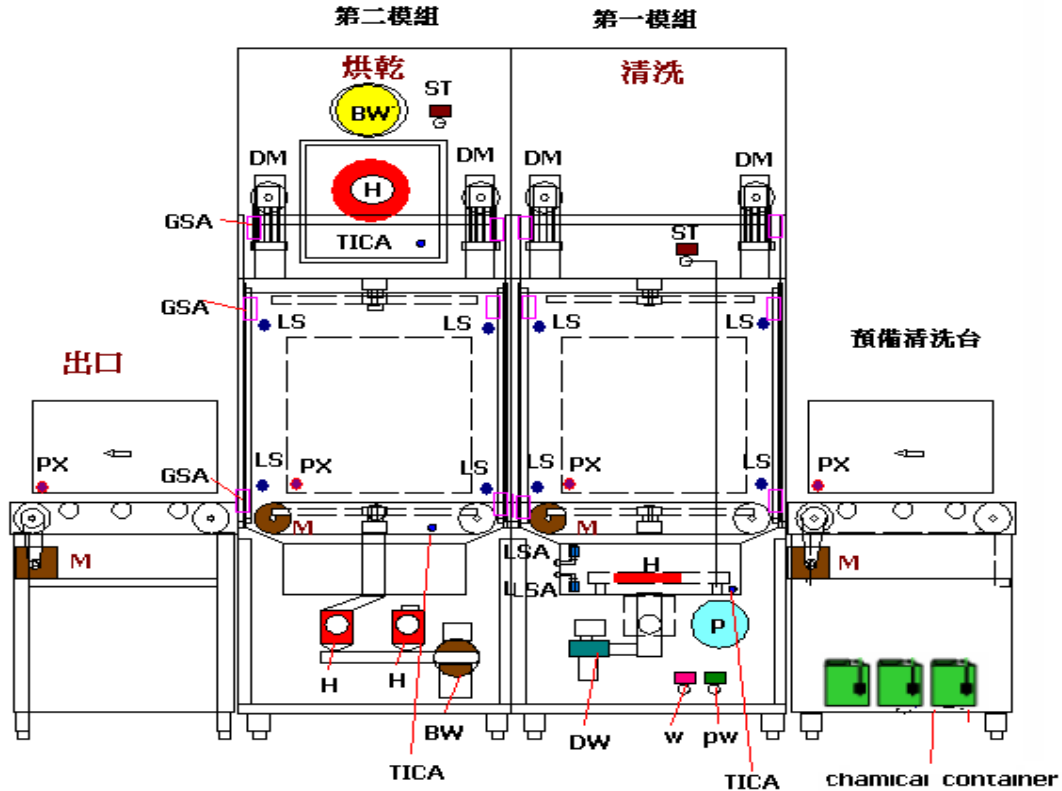
(2modules) with standard loading and unloading table

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Extensive variants of equipment

- Two-, module unit, vertical sliding doors
- Ultrasonic cleaning chamber
- Steam or electrical heating
- Monitoring of pump pressure, temperature, conductivity etc.
- Use of hollow jets with contact coupler to the basket (GMP rinse)
- High capacity HEPA filtered drying system
- Barcode reader for automatic basket identification
- Wide variety of baskets, customer specific manufacture
- Extensive automation, conveyors etc.
- Execution in ex-protection

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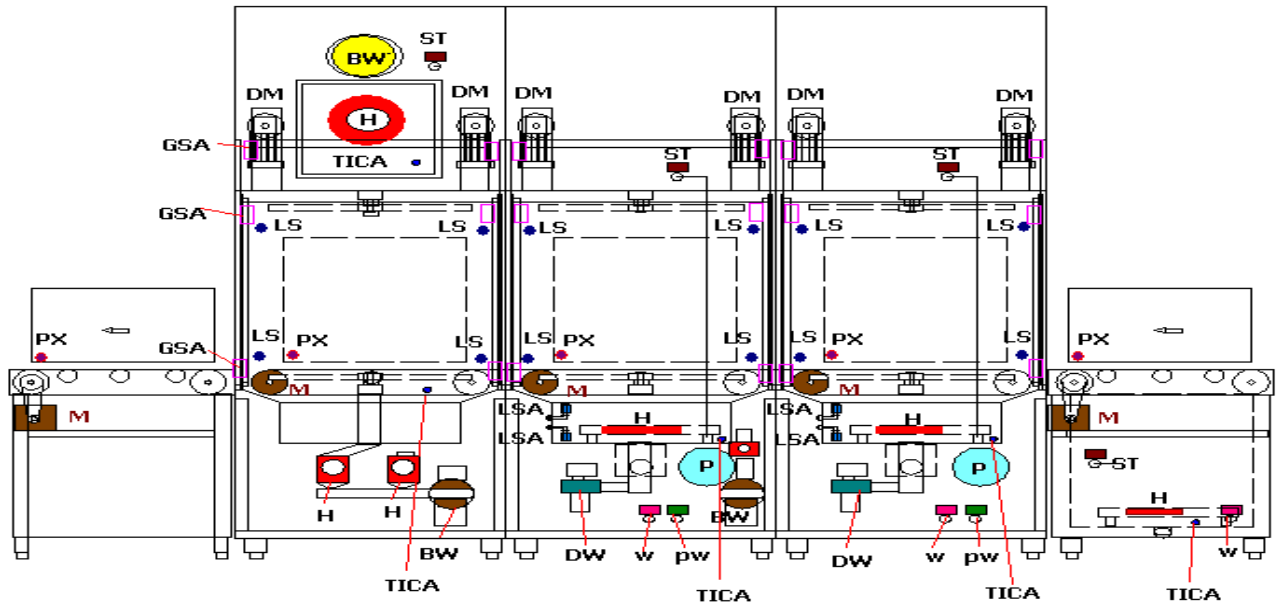
High Capacity Tunnel Washer LEADER® LD-570-3t

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
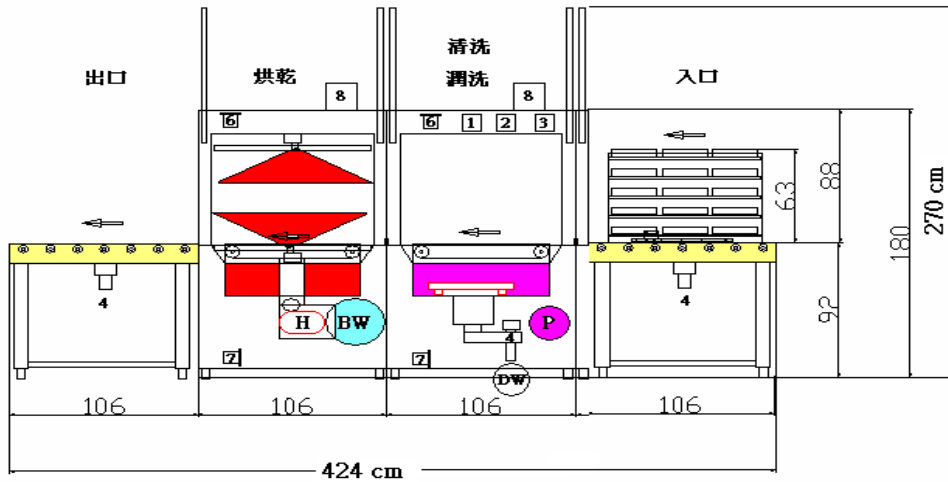
LEADER ULTRASONIC CO., LTD.

Ultrasonic Medical washer and dryers


LEADER® LD-570t series

Two-module unit, vertical sliding doors

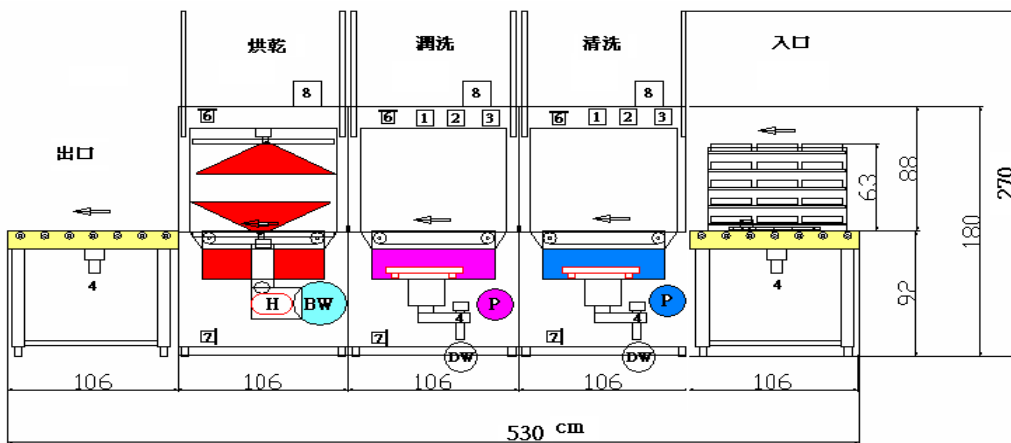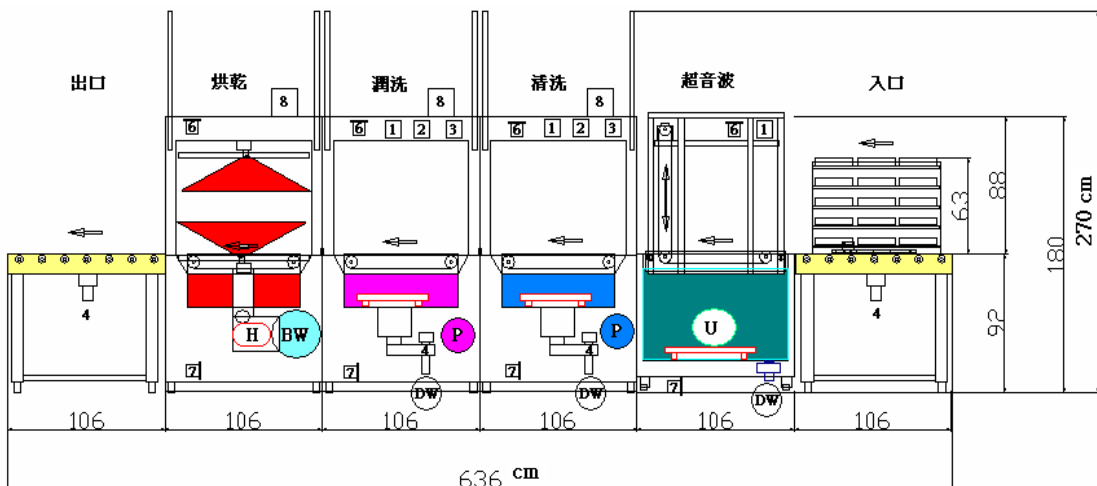
three module unit, vertical sliding doors

four module unit, vertical sliding doors

LEADER Medical Washer Disinfector

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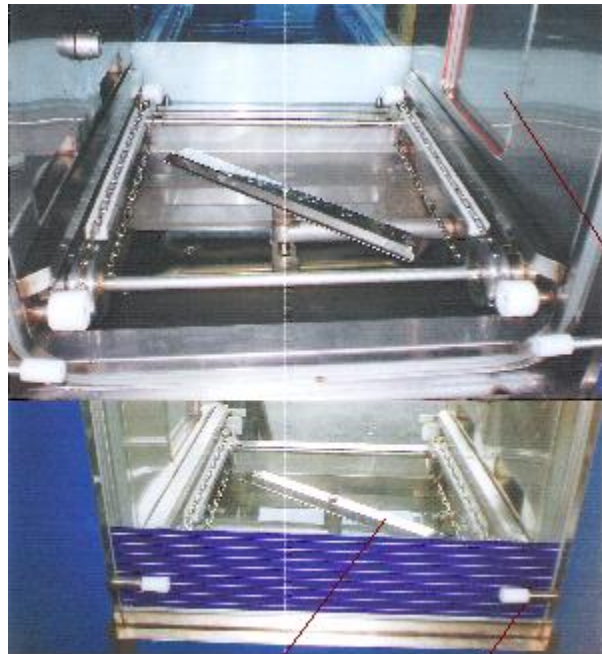LEADER ULTRASONIC CO., LTD.

**LEADER® MULTICHAMBER MEDICAL
TUNNEL WASHER
LEADER® LD-570T**

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